



Tender-Trode Plus™

Tubing & Cannula Holder

Features and Benefits

- Hydrocolloid is gentle on delicate skin
- Provides a barrier between adhesives and tender skin
- Eliminates irritation on skin from tape
- Fold over flap, reduces stress on tubes
- Open and close repeatedly for adjustment
- Ideal for use with nasal cannulas and feeding tubes

Patent #D506,547

VML095 Rev A



Tender-Trode Plus™

Tubing & Cannula Holder

5-Day Wear Time, **GUARANTEED!**



A10086



A10085

Cat No.	Size
<input type="checkbox"/> A10086	Mini
<input type="checkbox"/> A10085	Standard

**Remember... Warm and Hold,
Saturate to Remove**

Distributed By:
VERMED®
The Medical Sensor Company

9 Lovell Drive
Bellows Falls, VT 05101
Toll Free: 800-245-4025
www.vermed.com



Remember. . . Warm and Hold, Saturate to Remove

PROPER USE PROCEDURES

Prep



Gently warm for
30 to 45 seconds

Remove clear liner
from the base

Patient must be under professional supervision.

Apply



Apply to clean/dry skin

Hold in place for 60
seconds

Use



Lift rounded pull tab to
remove top liner and
fold over tubing/cannula

Lift blue tab to reposition
tubing/cannula

Discontinue use immediately if skin irritation occurs.

Remove



Lift blue tab to remove
tubing/cannula

Swab with water and
gently peel from skin
as you continue to swab
underside with water

Visit us @ www.vermed.com

Distributed By:
VERMED®
The Medical Sensor Company

Call us @ 1 (800) 245-4025

Insert for A100085 and A10086